# BRAUN

Nasal aspirator 1  
BNA100US

Contact Us  
Questions or Comments?  
Call us toll-free at: 1-800-327-7226  
E-mail: [consumerrelations@kaz.com](mailto:consumerrelations@kaz.com)

[www.braunnasalaspirator.com](http://www.braunnasalaspirator.com)

This product is manufactured under licence to the 'Braun' trademark. 'Braun' is a registered trademark of Braun GmbH, Kronberg, Germany.

## English

Please read all the instructions before using the Braun Nasal aspirator, especially this basic safety information.

- The nasal aspirator is designed to clean/extract nasal secretions only. **DO NOT** use the device for purposes other than those stated in these instructions for use.
- Keep out of reach of children under 12 years.
- **DO NOT** expose the Braun Nasal aspirator to direct sunlight.
- This device is designed for domestic use only. Consult a doctor if the child is suffering from an upper respiratory infection.
- **DO NOT** use other nozzles than those supplied with this device.
- **DO NOT** use alcohol-based cleaners on the collection cup.

## Limited two year warranty

You should first read all Instructions before attempting to use this product.

A. This 2 year limited warranty applies to repair or replacement of product found to be defective in material or workmanship. This warranty does not apply to damage resulting from commercial, abusive, unreasonable use or supplemental damage. Defects that are the result of normal wear and tear will not be considered manufacturing defects under this warranty. **KAZ IS NOT LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES OF ANY NATURE, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS LIMITED IN DURATION TO THE DURATION OF THIS WARRANTY.** Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you also may have other rights which vary from jurisdiction to jurisdiction. This warranty applies only to the original purchaser of this product from the original date of purchase.

- B. At its option, Kaz will repair or replace this product if it is found to be defective in material or workmanship.
- C. This warranty does not cover damage resulting from any unauthorized attempts to repair or from any use not in accordance with the instruction manual.

Call us toll-free at 1-800-327-7226 or e-mail: [consumerrelations@kaz.com](mailto:consumerrelations@kaz.com). Or visit our website at: [www.braunnasalaspirator.com](http://www.braunnasalaspirator.com). Please be sure to specify a model number.

**NOTE: IF YOU EXPERIENCE A PROBLEM, PLEASE CONTACT CONSUMER RELATIONS FIRST OR SEE YOUR WARRANTY. DO NOT RETURN THE PRODUCT TO THE ORIGINAL PLACE OF PURCHASE. DO NOT ATTEMPT TO OPEN THE DEVICE HOUSING, DOING SO MAY VOID YOUR WARRANTY AND CAUSE DAMAGE TO THE PRODUCT OR PERSONAL INJURY.**

## Specifications

Model	BNA100US
Operating temperature	50 – 104 °F
Storage temperature	-13 – 131 °F
Operation & storage relative humidity	≤95 % R.H. non-condensing
Power source	Alkaline battery: 2 x AA (LR6) 1.5V
Battery life	1000 operations - 3 hours
Service life	5 years
Warranty	2 years

Equipment with  
type BF applied parts

See instruction for use

Operating temperature

Storage temperature

Subject to change without notice.

**IP22**

Protected against solid foreign objects of .49 in. diameter and larger.  
Protected against water drops falling vertically when the device is tilted up to 15°.

Portable and mobile RF communications equipment can affect medical electrical equipment.

This product contains batteries and recyclable electronic waste. To protect the environment, do not dispose of it in household waste, but take it to an appropriate local collection center provided in your country.

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© 2018, All rights reserved.
Kaz USA, Inc., a Helen of Troy Company